

Follistim *Pen*TM

For use only with
Follistim[®] AQ Cartridge
(follitropin beta injection)



INSTRUCTIONS FOR USE



IMPORTANT NOTICE:

Please read this safety information first.

1. Follistim Pen™ is a precision device. It is very important that you read and follow all directions for its use. As with all injection systems, failure to do so could result in an incorrect dose being given.
2. Injecting cold drug is likely to cause discomfort. Therefore, it is recommended you allow the drug to reach room temperature before taking the injection. The needle should not be attached until just before the injection.
3. Follistim® AQ Cartridge (follitropin beta injection) is NOT the same formulation as Follistim® in the freeze-dried powder presentation.
4. When injecting beneath the skin, always fully depress the Injection Button for a full 5 seconds. During these 5 seconds, leave the needle in the skin to ensure delivery of the full dose of Follistim® AQ Cartridge.
5. Follistim Pen™ is designed for use by one person only and should not be shared with others.
6. The BD Micro-Fine™ Pen Needle is for use only with Follistim Pen™. Each BD Micro-Fine™ Pen Needle is for one injection only.
7. Follistim Pen™ is not recommended for the blind or visually impaired user without the assistance of an individual with good vision, trained in the proper use of the device.
8. It is advisable to keep a record of every injection that has been taken from a Follistim® AQ Cartridge. This will allow you to calculate the daily amount of medication remaining in the cartridge by subtracting the injected amount from the 300 IU and/or 600 IU as indicated on the cartridge label.
9. Your doctor will decide on the dose of Follistim® AQ Cartridge to be given. This dose may be increased or decreased as your treatment progresses depending on your individual type of treatment. If you have previously used gonadotropins, your dose may be different with Follistim® AQ Cartridge. Again, your doctor will decide the dose based upon your medical history.

INTRODUCTION

BEFORE USING FOLLISTIM PEN™ FOR THE FIRST TIME, READ THESE INSTRUCTIONS CAREFULLY. THEY ARE DIVIDED INTO SECTIONS. EACH SECTION HAS A NUMBER OF STEPS.

KEEP THIS BOOKLET IN A SAFE PLACE AND REFER TO IT WHEN QUESTIONS ARISE.

The main actions contained in this booklet that must be learned are:

1. Loading the Follistim Pen™
2. Dialing the prescribed dose
3. Using the Follistim Pen™ to give yourself an injection of Follistim® AQ Cartridge

The Follistim Pen™ is a very precise injection device. It is easy to use and convenient. The special features allow you to inject Follistim® AQ Cartridge safely and effectively.


Follistim Pen™ can be set to any dose from 50 International Units (IU) up to 450 IU, in marked increments. The Dosage Scale on the Pen has easy-to-read numbers and audible clicks to help you set the correct dose. Incorrect settings are easily corrected without loss of drug.




- *Follistim® AQ Cartridges and BD Micro-Fine™ Pen Needles may only be used with Follistim Pen™.*
- *Do not separate these three parts of the Follistim Pen™ until told to do so in the instructions.*



Follistim Pen™ Parts

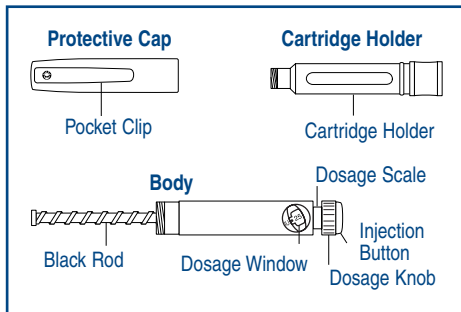

Follistim® AQ Cartridge and BD Micro-Fine™ Needle Unit Parts

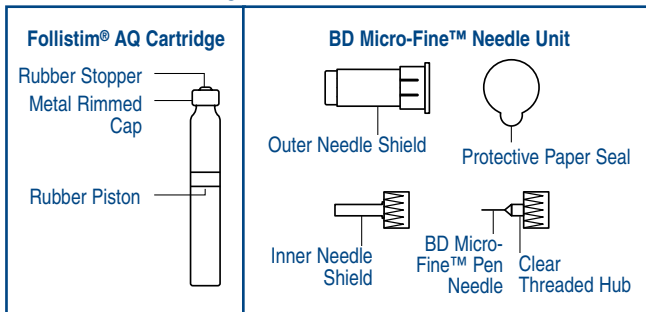

2



FOLLISTIM PEN™ AND THE FOLLISTIM® AQ CARTRIDGE

General Information

Follistim® AQ solution for injection in cartridges has been developed for use with the Follistim Pen™ and should be given subcutaneously (beneath the skin).

The daily dose (IU of Follistim® AQ Cartridge to be injected each day) will be determined by your healthcare professional. A Treatment Diary is located in the back of this booklet. Use it to keep a record of your Follistim® AQ Cartridge injections.

During your treatment cycle, your healthcare professional will advise you of the best time of the day to inject yourself. Try to inject yourself at the same time of day every day.

WHAT YOU WILL NEED FOR GIVING YOURSELF THE INJECTION

- Follistim Pen™, a Follistim® AQ Cartridge, the BD Micro-Fine™ Pen Needle provided with the cartridge.
- Alcohol, cotton balls or alcohol pads/swabs, sterile gauze, antibacterial soap and a special safety container to discard the used needles, cartridges and/or other supplies.




3



USING THE FOLLISTIM PEN™

Follow these steps:

- Before you use the Follistim Pen™ for the first time
 - When you replace the cartridge
- Note: Always wash your hands thoroughly with antibacterial soap and water before you use the Follistim Pen™ or when you replace the cartridge.

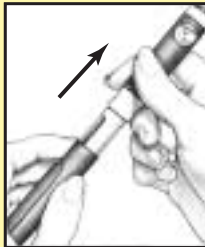

1. While holding the Pen Body firmly with one hand, pull off the Protective Cap with your other hand. Put the cap aside on a clean, dry surface.




2. Unscrew the entire Pen Body from the Cartridge Holder. Place the Cartridge Holder and the Pen Body aside on a clean, dry surface.



4



LOADING THE FOLLISTIM PEN™

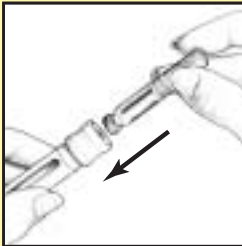

3. Take a Follistim® AQ Cartridge out of its package. Do not use the Follistim® AQ Cartridge if the medicine contains particles or it is not clear. Make sure the medicine is at room temperature before using. Clean the Rubber Stopper on the cartridge with an alcohol pad. Pick up the Cartridge Holder and place the cartridge into the Cartridge Holder. Insert the **Metal Rimmed Cap** end first.



4. Pick up the Pen Body and lower it into the Cartridge Holder. The black rod must press against the Rubber Piston on the cartridge. Screw the Pen Body fully onto the Cartridge Holder. Make sure there is no gap between the Pen Body and the Cartridge Holder. The arrow ▲ on the Cartridge Holder should point to the middle of the yellow alignment mark ■ on the blue Pen Body.



5



ATTACHING THE BD MICRO-FINE™ PEN NEEDLE



5. You must use a new BD Micro-Fine™ Pen Needle with each injection. Never reuse a needle. Attach a BD Micro-Fine™ Pen Needle after you make sure there is a Follistim® AQ Cartridge in the Cartridge Holder. Clean the open end of the Cartridge Holder with an alcohol pad.



6. Pick up your BD Micro-Fine™ Pen Needle that is in an Outer Needle Shield. Peel off the Protective Paper Seal. Do not touch the needle or place an open needle on any surface.




7. Hold the Outer Needle Shield firmly in one hand while holding the Cartridge Holder firmly in the other hand. Push the end of the Cartridge Holder into the Outer Needle Shield. Screw them tightly together. Place your Follistim Pen™ with the attached needle, flat on a clean, dry surface.



6



PREPARING THE INJECTION SITE

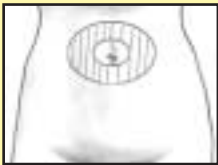

8. The best place for injection is in the abdomen, which is the stomach area below the belly button (navel) or in the upper leg. Your healthcare provider can show you other places where you can inject Follistim® AQ Cartridge. Change your injection site a little bit with each injection to lower your chances for skin reactions.

9. Use a swab moistened with alcohol to clean the skin area where the needle will enter to remove any surface bacteria. Clean about two inches around the injection site where the needle will be inserted. Let the alcohol dry on your skin for at least one minute before injecting the medicine.



7



CHECKING THE FLOW OF FOLLISTIM® AQ CARTRIDGE THROUGH THE BD MICRO-FINE™ PEN NEEDLE



10. Pull the Outer Needle Shield gently, leaving the Inner Needle Shield in place (covering the needle, which is now attached to the Pen). **Do not throw the Outer Needle Shield away.** You will need it to dispose of the needle after injecting the medicine. (See REMOVING & DISPOSING OF THE BD MICRO-FINE™ PEN NEEDLE.)

Important:


- Do not touch the needle or leave it uncapped without Inner Needle Shield so that it remains sterile.
- Use a new, sterile BD Micro-Fine™ Pen Needle for each injection. Only use the BD Micro-Fine™ 0.33 mm x 12.7 mm (29G) Pen needles as supplied with the Follistim® AQ Cartridge.



8



CHECKING THE FLOW OF FOLLISTIM® AQ CARTRIDGE THROUGH THE BD MICRO-FINE™ PEN NEEDLE - CONTINUED

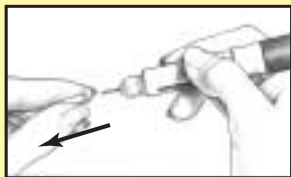

11. Carefully remove the Inner Needle Shield and discard it. **Do not touch the needle or let it touch any surface while uncapped.**



12. Hold the Follistim Pen™ with the needle pointing upwards. Tap the Cartridge Holder gently with your finger to help air bubbles rise to the top of the needle. The small amount of air bubble will not affect the amount of medicine you receive. Look for a droplet forming at the tip of the needle. If you see a droplet, go to Step 13. If you do not see a droplet, continue with this step.

If you do not see a droplet at the tip of the needle:

- a. Dial the Dosage Knob until you hear one click. With the needle pointing upwards, push in the Injection Button.
- b. Look for the droplet.
- c. If you still do not see a droplet, **repeat Step a** (above) until you see droplet. You must make sure you see a droplet of medicine or you may not inject the right amount of medicine.

Important:

- Always check the drug flow by following the directions in Step 12. This will ensure that the correct dose of Follistim® AQ Cartridge is injected. If you do not check drug flow, you may receive less drug than the dose that you dialed.



9



DIALING THE PRESCRIBED DOSE



13. Your Follistim® AQ Cartridge will contain either 300 IU or 600 IU. Record the Follistim® AQ Cartridge content in your Follistim Pen™ Treatment Diary (refer to CHECKING THE DRUG LEVEL for more information). For doses of 50 IU up to 450 IU, turn the Dosage Knob until the dot beside the correct number on the Dosage Scale is sitting in the middle of the Dosage Window.

14. If by mistake you dial past the correct number, **do not try to turn the Dosage Knob backward to fix the mistake.** Continue to turn the Dosage Knob in the same direction past the 450 IU mark, as far as it will turn. The Dosage Scale must move freely. Push the Injection Button in all the way. Start to dial again starting from “0” upwards.

By following these directions, you will not lose any Follistim® AQ Cartridge.

Important:

- If you do turn the Dosage Knob backward to correct the mistake, it will not hurt the Pen, but you will lose some drops of Follistim® AQ Cartridge from the tip of the needle.
- Never dial your dose or try to correct a dialing mistake when the needle is still in your skin as this may result in your receiving an incorrect dose.



10



USING THE FOLLISTIM PEN™ TO GIVE YOURSELF AN INJECTION OF FOLLISTIM™ AQ CARTRIDGE

Once you have set the Follistim Pen™ to the correct dose, you are ready for your injection.



15. Pinch the already swabbed area of the skin between two fingers. With the other hand, insert the entire BD Micro-Fine™ Pen Needle straight into the skin. Press the injection button all the way in to make sure you give yourself a full injection. Wait for five seconds before pulling the needle out of the skin. The middle of the Dosage Window should display a dot next to the “0”. If the injection button does not push in all the way, and the number in the Dosage Window does not read “0”, it means there is not enough medication left in the cartridge. The number in the Dosage Window will give you the amount of medicine needed to complete your dose. Write this

number down. This will be the number you dial for the completion of your dose. Start over with a new Follistim® AQ Cartridge and a new needle and follow all the instructions up to this step. Make sure you choose a different injection site to complete your dose of Follistim® AQ Cartridge.



16. Pull out the BD Micro-Fine™ Needle and firmly press down on the injection site with an alcohol swab. Use the BD Micro-Fine™ Pen Needle for one injection only.

Important:

- If the dot next to the “0” is not in the middle of the Dosage Window and you cannot push the Injection Button all the way in, do not try to force down the button. Your Follistim® AQ Cartridge is probably empty. This means you have not received your full dose. Do not adjust the setting on the Dosage Scale. Follow the instructions for ‘IF THERE IS NOT ENOUGH FOLLISTIM® AQ IN THE CARTRIDGE’.



11



REMOVING AND DISPOSING OF THE BD MICRO-FINE™ PEN NEEDLE



17. Place the Outer Needle Shield on a flat table surface with the opening pointing up. The opening of the Outer Needle Shield is the wider end with the rim. Without holding on to the Outer Needle Shield, carefully insert the needle (attached to the Follistim Pen™) into the opening of the Outer Needle Shield and push down firmly. The Outer Needle Shield should now be attached to the Cartridge Holder and cover the needle.



18. Grip the Outer Needle Shield and use it to unscrew the needle from the Cartridge Holder. Your healthcare provider can advise you

on how to obtain a special container for proper needle disposal. Safely, dispose of the Outer Needle Shield with the used needle right away. Do not throw away in a trash can. If there is Follistim® AQ Cartridge medicine left for more injections, put the Pen Cap back on the Pen Body and store your Follistim Pen™ in a safe place in the refrigerator (**not in the freezer**) or at room temperature. **Never store the Follistim Pen™ with a needle attached to it.**

Important:


- Use the BD Micro-Fine™ Pen Needle for one injection only. Dispose of the used needle inside the Outer Needle Shield immediately in a properly secured container as instructed by your healthcare provider.
- Always unscrew the needle by following the directions in Steps 17 and 18 before you put the Follistim Pen™ away.
- Never store the Follistim Pen™ with a needle attached to it. If you store the Follistim Pen™ with the needle attached, the drug could leak out and there is risk of contamination.
- Never leave needles where others can pick them up.



12



REMOVING THE EMPTY FOLLISTIM® AQ CARTRIDGE

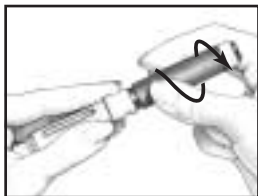

19. Before you remove the empty Follistim® AQ Cartridge from the Follistim Pen™, unscrew the needle by following Steps 17 and 18. Unscrew the Pen Body from the Cartridge Holder.



20. Put the Pen Body down on a clean, dry surface and remove the empty Follistim® AQ Cartridge from the Cartridge Holder. Safely, dispose of the empty Follistim® AQ Cartridge right away in the same special container that you used for the needle disposal. Do not put the cartridge in a trash can.



13



REMOVING THE EMPTY FOLLISTIM® AQ CARTRIDGE - CONTINUED

Now, you may either:

- Put the Follistim Pen™ back together without a Follistim® AQ Cartridge and store for your next injection.
- OR**
- Insert a new Follistim® AQ Cartridge into the Cartridge Holder, put your Follistim Pen™ back together, and store safely in the refrigerator or at room temperature.

Important:

- Upon dispensing, the product may be stored by the patient at 2–8°C (36–46°F) until the expiration date, or at 25°C (77°F) for 3 months or until expiration date, whichever occurs first. Do not freeze. Protect from light.
- Once the Rubber Stopper of the Follistim® AQ Cartridge has been pierced by a needle, the product can only be stored for a maximum of 28 days at 2–25°C (36–77°F).



CHECKING THE DRUG LEVEL

When you are advised on the number of prescribed doses which can be extracted from the full unused Follistim® AQ Cartridge (follitropin beta injection), then do not use the cartridge beyond the advised number of doses. Otherwise, you will run the risk that there will not be enough volume of drug for your prescribed dose.

Starting with the initial injection, you should begin using your Follistim Pen™ Treatment Diary (located in the back of this booklet) as follows:

- A. Record the Follistim® AQ Cartridge content. This will either be 300 or 600 IU depending on what your doctor has prescribed for you.
- B. Record the daily dose you have been prescribed for your injection.
- C. Subtract your Day 1 dose from the Follistim® AQ Cartridge content (300 or 600 IU) (see example below). This will give you the remaining Follistim® AQ Cartridge content.



CHECKING THE DRUG LEVEL - CONTINUED

After Day 1, continue to record your daily injections in your Treatment Diary as follows:

- D. Place the number recorded on Day 1 in the remaining Follistim® AQ Cartridge contents box in the Day 2 Follistim® AQ Cartridge content box.
- E. Subtract your Day 2 dose from the Follistim® AQ Cartridge content you just recorded in Step D. This will give you the remaining Follistim® AQ Cartridge content. Again, record this number in the correct box.

Date	Daily Dose to be Used (IU/day)	Follistim® AQ Cartridge Content	Remaining Follistim® AQ Cartridge Content
mm/dd/yy	150	600	450
mm/dd/yy	150	450	300
mm/dd/yy	150	300	150

If there is any doubt that there is not enough drug in the Follistim® AQ Cartridge for your prescribed dose, see section 'IF THERE IS NOT ENOUGH FOLLISTIM® AQ IN THE CARTRIDGE' on page 17.



IF THERE IS NOT ENOUGH FOLLISTIM® AQ IN THE CARTRIDGE

If you realize before you inject that you do not have enough drug remaining in your Follistim® AQ Cartridge for your complete dose, proceed to Step 1 (page 18).

OR

If you have already inserted the needle at the injection site and the Injection Button will not push in all the way, then there is not enough drug contained in the cartridge. The number in the Dosage Window will not read "0" – this means that there is not enough drug remaining in the Follistim® AQ Cartridge to complete your dose. In this case, proceed to Step 2 (page 19).

Important:

- **When there is not enough drug contained in the Follistim® AQ Cartridge to complete your injection, the Injection Button will not push in all the way and the number in the Dosage Window will not read "0". If this occurs, proceed with the steps that follow.**



IF THERE IS NOT ENOUGH FOLLISTIM® AQ IN THE CARTRIDGE - CONTINUED

1. If you realize you do not have enough drug remaining in your Follistim® AQ Cartridge, you have 2 options:

Option #1: Remove the Follistim® AQ Cartridge as outlined in Steps 19 and 20. Insert a new cartridge into the Follistim Pen™ (see Steps 3 and 4). Continue with Steps 5 – 18 for your injection.

Option #2: Dial your dose and inject the remaining content in the Follistim® AQ Cartridge. The Dosage Knob Injection Button will not push in all the way and the Dosage Window number will not read “0” but will read the number of units you will need to complete your prescribed dose. Remember to write down the number of units needed to complete your dose. Remove the needle and dispose of it properly as outlined in Steps 17 and 18. Using the Dosage Knob, reset the Dial Window to “0” by turning the Dosage Knob past the 450 IU mark as far as it will turn and push the Injection Button in all the way. Insert a new cartridge into the Follistim Pen™ and attach a new BD Micro-Fine™ Pen Needle (see Steps 3 – 12). Dial to the number of units you have written down to complete your prescribed dose. Prepare a different injection site and inject the remaining drug to complete your dose (refer to USING THE FOLLISTIM PEN™ TO GIVE YOURSELF AN INJECTION OF FOLLISTIM® AQ CARTRIDGE).



IF THERE IS NOT ENOUGH FOLLISTIM® AQ IN THE CARTRIDGE - CONTINUED

2. If you have already inserted the needle at the injection site, inject the remaining content in the Follistim® AQ Cartridge. The Injection Button will not push in all the way and the number in the Dosage Window will not read “0” but will read the number of units you will need to complete your prescribed dose. Wait 5 seconds before withdrawing the needle from your skin and gently apply pressure to the injection site with an alcohol pad. Dispose of the used needle properly as outlined in Steps 17 and 18. Remember to write down the number of units needed to complete your dose. Using the Dosage Knob, reset the Dial Window to “0” by turning the Dosage Knob past the 450 IU mark as far as it will turn and push the Injection Button in all the way. Insert a new cartridge into the Follistim Pen™ and attach a new BD Micro-Fine™ Pen Needle (see Steps 3 – 12). Dial to the number of units you have written down to complete your prescribed dose. Prepare a different injection site and inject the remaining drug to complete your dose (refer to USING THE FOLLISTIM PEN™ TO GIVE YOURSELF AN INJECTION OF FOLLISTIM® AQ CARTRIDGE).

Important:

- **Always** remove the needle from the Follistim Pen™ after you have completed your injection. If you store the Pen with the needle attached, the drug could leak out and there is risk of contamination. Dispose of the used needle in a properly secured container as instructed by your healthcare provider.



SOLVING PROBLEMS

PROBLEM	POSSIBLE CAUSES	WHAT TO DO
No drug is coming out while checking the flow.	Is the Follistim® AQ Cartridge empty?	Change to a new cartridge – See page 18.
No drug is coming out while injecting.	Is the BD Micro-Fine™ Pen Needle blocked?	Press the Injection Button in all the way so the dial goes back to “0”.
	Is the Follistim® AQ Cartridge empty?	Change to a new cartridge – See page 18.
The Dosage Knob does not go back to “0” while you are injecting.	Is the Follistim® AQ Cartridge empty?	Change to a new cartridge – See page 18.
	Are you in the middle of correcting the dose?	Change to a new BD Micro-Fine™ Pen Needle – See page 6.
	Is the BD Micro-Fine™ Pen Needle blocked?	Change to a new BD Micro-Fine™ Pen Needle – See page 6.



SOLVING PROBLEMS - CONTINUED

PROBLEM	POSSIBLE CAUSES	WHAT TO DO
The Dosage Knob is hard to turn.	Is dust or dirt in the way?	Dial past the highest dose setting on the scale. Clean the Follistim Pen™ by wiping all exposed surfaces with a clean, damp cloth.
You are concerned that you can turn the Dosage Knob to the next number without clicking.	This is not a problem.	The system is in the reset mode. The Injection Button and Dose Knob must be pushed all the way down to zero to re-engage the mechanism and the correct dose can now be set. A click will be heard for each setting in the viewing window.
The Injection Button spins freely.	This is not a problem.	This is part of the design of the Follistim Pen™.
After your last injection, a remaining volume may be left in the cartridge in addition to the normal quantity of drug dispensed.	The cartridge contains extra volume for checking the drug flow.	This is not a problem.



SOLVING PROBLEMS - CONTINUED

PROBLEM	POSSIBLE CAUSES	WHAT TO DO
The Pen Body will not screw tightly into the Cartridge Holder.	Is something in the way?	Take out the Follistim® AQ Cartridge and check the Cartridge Holder to see if anything is in the way. If nothing is there, carefully repeat the steps on page 4.
You cannot get the Cartridge out of the Follistim Pen™.	Is the BD Micro-Fine™ Pen Needle attached?	Remove the BD Micro-Fine™ Pen Needle from the Follistim Pen™ and dispose of properly (See page 12). Unscrew the Cartridge Holder from the Pen Body and take out the cartridge (See page 13).
Some of the drug is dripping out of the BD Micro-Fine™ Pen Needle when you withdraw it out of your skin.	Did you take the BD Micro-Fine™ Pen Needle out of your skin too soon?	Always wait 5 seconds after you push the Injection Button before you withdraw the BD Micro-Fine™ Pen Needle from your skin.



SOLVING PROBLEMS - CONTINUED

PROBLEM	POSSIBLE CAUSES	WHAT TO DO
You are not sure how much drug is left in the Cartridge and you do not want to start an injection and then find out that there is not enough drug.	Have you kept good records of your doses?	Write down every injection you have taken from the Follistim® AQ Cartridge. Follow the instructions on the Follistim® AQ Cartridge label. Talk to your healthcare professional if you are concerned.
The BD Micro-Fine™ Pen Needle is left on the Follistim Pen™.	Have you missed any of the instructions?	Dispose of the BD Micro-Fine™ Pen Needle in a properly secured container as instructed by your healthcare professional. Change to a new Follistim® AQ Cartridge, and a new BD Micro-Fine™ Pen Needle.



MORE INFORMATION

Caring for the Follistim Pen™

1. Clean all exposed surfaces of the Follistim Pen™ with a clean, damp cloth. Never wash it in water, detergent or strong medical cleaners.
2. Handle the Pen carefully to avoid causing damage. You could damage the Pen by dropping it or handling it roughly.
3. Keep the Pen away from dust and dirt.
4. If the Pen breaks, do not try to fix it yourself. Contact your healthcare provider.

Storing the Follistim Pen™

1. Store refrigerated, 2–8°C (36–46°F) until dispensed. Upon dispensing, the product may be stored by the patient at 2–8°C (36–46°F) until the expiration date, or at 25°C (77°F) for 3 months or until expiration date, whichever occurs first. Do not freeze. Once the Rubber Stopper of the Follistim® AQ Cartridge has been pierced by a needle, the product can only be stored for a maximum of 28 days at 2–25°C (36–77°F).
For proper storage of unused Follistim® AQ Cartridges, please see the package insert enclosed with Follistim® AQ Cartridges.
2. Follistim Pen™ with a Follistim® AQ Cartridge should be protected from light.
3. Do not use past the indicated expiration date on the Follistim® AQ Cartridge.
4. Keep the Follistim Pen™ containing a Follistim® AQ Cartridge out of the reach and sight of children.
5. After your treatment is finished, store the Follistim Pen™ device as instructed by your healthcare provider.



PRECAUTIONS

- Do not share your Follistim Pen™ with another person.
- The Follistim Pen™ is not recommended for the blind or visually impaired user without the assistance of an individual with good vision, trained in the proper use of the device.
- The Follistim Pen™ is only indicated for use with a Follistim® AQ Cartridge when prescribed for subcutaneous injection.



Care while using the BD Micro-Fine™ Pen Needle

- Only attach the BD Micro-Fine™ Pen Needle when you are ready to inject. Always remove the needle from the Follistim Pen™ immediately and dispose of it properly in its Outer Needle Shield after you complete your injection.
- The Pen Needle Unit is sterile. To avoid contaminating the needle after opening, do not place it on any surface or touch exposed parts.
- Before attempting to replace a Follistim® AQ Cartridge, be sure that a BD Micro-Fine™ Pen Needle is not attached to the Follistim Pen™.
- Never dial your dose or attempt to correct a dialing error with the needle in your skin, as this may result in an incorrect dose.
- Proper procedures should be used for the disposal of the used needles (fixed in the Outer Needle Shield), “empty” cartridges, and leftover medication. Consult your healthcare provider on the proper procedures for disposal.
- The Follistim Pen™ is intended for self-injection of Follistim® AQ Cartridge. If you are giving an injection to another person, **be very careful when removing the needle from the skin.** Accidental needle sticks can transmit infectious diseases.



CUSTOMER SERVICE

To keep the quality of our products high, Organon USA Inc. is very interested in what you have to say about the Follistim Pen™.

If you have any complaints or problems with how a drug is affecting you, you can do two things:

1. Talk to your healthcare professional.
2. Report your concerns to Customer Service, Organon USA Inc., 1-800-241-8812.

If you have any complaints or suggestions about the Follistim Pen™, please write, call or fax Customer Service, Organon USA Inc., 56 Livingston Avenue, Roseland, NJ 07068
Phone: 1-800-241-8812 Fax: 1-800-509-3915

Follistim Pen™

Manufactured for Organon USA Inc. by BD (Becton, Dickinson and Company)